




## Phonak Terra BTE-UP/Phonak Terra+ BTE-UP

### Standard Earhook HE11 2 cm<sup>3</sup> coupler data

ANSI / ASA S3.22-2014 (R2020)  
IEC 60118-0: 2015 / IEC 60118-0: 2022


#### Output sound pressure level

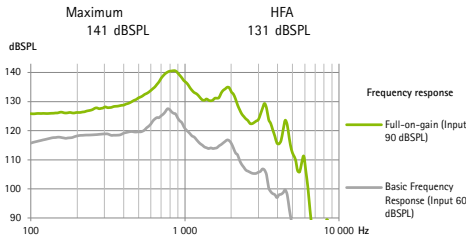


### Standard Earhook HE11 680 2 cm<sup>3</sup> coupler data

ANSI / ASA S3.22-2014 (R2020)  
IEC 60118-0: 2015 / IEC 60118-0: 2022

#### Output sound pressure level

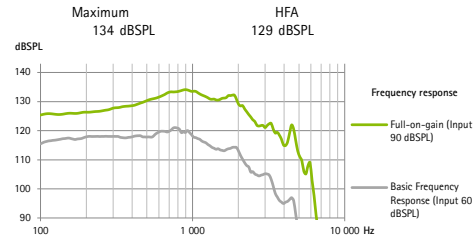


#### Acoustic gain

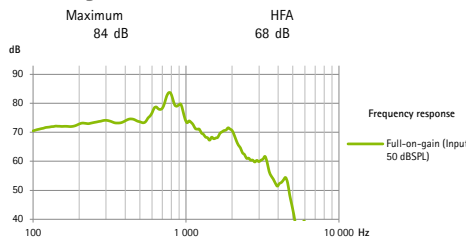


| Frequency range              | <100 Hz - 4700 Hz |        |         |         |
|------------------------------|-------------------|--------|---------|---------|
|                              | 500 Hz            | 800 Hz | 1600 Hz | 3200 Hz |
| Total harmonic distortion    | 5,0%              | 1,0%   | 1,0%    | 1,0%    |
| Battery current              | 3,3 mA            |        |         |         |
| Expected operating time*     | 188 h             |        |         |         |
| Equivalent input noise level | 19 dB SPL         |        |         |         |

#### Acoustic gain

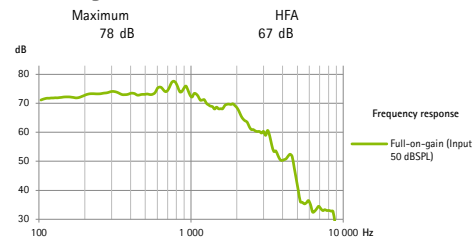


| Frequency range              | <100 Hz - 4700 Hz |        |         |         |
|------------------------------|-------------------|--------|---------|---------|
|                              | 500 Hz            | 800 Hz | 1600 Hz | 3200 Hz |
| Total harmonic distortion    | 3,0%              | 1,0%   | 1,0%    | 1,0%    |
| Battery current              | 2,8 mA            |        |         |         |
| Expected operating time*     | 221 h             |        |         |         |
| Equivalent input noise level | 19 dB SPL         |        |         |         |

#### Induction coil sensitivity

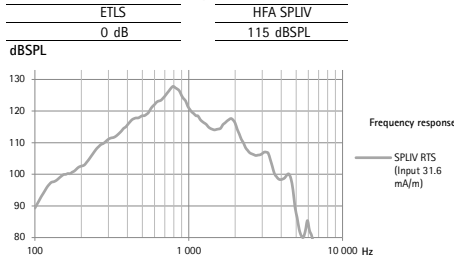

#### Induction coil sensitivity

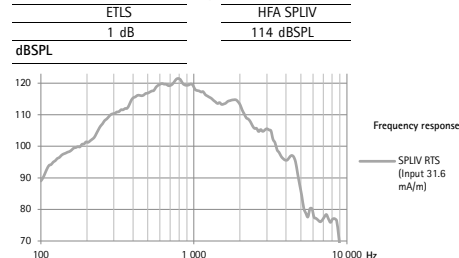

#### General test information

- Supply voltage 1.3 V / impedance 6.2 Ω
- Specific measurement settings are used. RTS adjustment with volume control
- The device is operating in linear mode
- Low-level expansion is active
- All data obtained are measured with Phonak Target measurement settings
- The latency of the audio signal determined according an internal standard is 6.2 ms

#### Warnings

- ⚠ This hearing instrument has an output sound pressure level that can exceed 132 dB SPL. Special care should be taken when fitting this instrument as there is a risk of impairing the residual hearing of the user.
- ⚠ Changes or modifications to the hearing aid that are not explicitly approved by the manufacturer are not permitted. Such changes may damage the ear or the hearing aid.
- ⚠ The developed SPL in the ears of children can be substantially higher than in average adults. RECD measured to correct target of fitter OSPL90 is recommended.

\* Battery performance depends on active features, the use of wireless accessories, hearing loss, battery age, sound environment and earpiece. Note that for non-rechargeable ZnAir batteries operating time might vary depending on the battery model.



Sonova AG · Laubisrütistrasse 28  
CH-8712 Stäfa · Switzerland  
www.phonak.com

A Sonova brand

